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Sterile disposable wipes for the periocular area treatment

Indications

KLIMÌ MED is a soft disposable sterile wipe, containing hyaluronic acid, which provides detergent, emollient, soothing and decongestant activities.

KLIMÌ MED is a medical device indicated to remove secretions or squamous residues from the eyelids and eyelashes caused by inflammatory and/or infectious phenomena.

KLIMÌ MED is also recommended with medical therapy for restoring and maintaining the physiological conditions of the periocular area in the case of allergic, inflammatory or infectious diseases of the eye and its annexes, particularly before and after ocular surgery.

KLIMÌ MED is perfume and paraben-free.

Composition

Sodium hyaluronate, chlorhexidine digluconate, ammonium glycyrrhizinate, disodium phosphate dodecahydrate, sodium phosphate monobasic monohydrate, disodium edetate, polysorbate 20, glycerin, sodium hydroxymethylglycinate, citrus aurantium amara flower distillate, water.

Presentation

1 box containing 14 disposable sterile wipes.

1 leaflet.

Instructions for use

Wash your hands thoroughly and open the sachet to pull out the moist wipe.

Close the eye, place the wipe over the eyelid and lashes and gently massage until the entire eye area is thoroughly cleansed.

In case of persistent secretions, you can heat the closed sachet in warm water for a few minutes, use the wipe as a warm compress and massage gently.

In the case of inflammation, it is possible to cool the closed sachet in a refrigerator for a few hours and place the open wipe on the closed eye for a few minutes as a cold compress.

Contraindications

Hypersensitivity to the product or to any of its components.

Precautions and warnings

 Do not use **KLIMÌ MED** if the container is opened or damaged because the functionality and safety of the product are guaranteed by the integrity of original packaging.

KLIMÌ MED is a disposable sterile wipe, therefore it is intended to be used only once to avoid unintentional contamination of the periocular surface.

It is important to use always a separate wipe for each eye in order to avoid possible cross-contamination.

KLIMÌ MED contains **chlorhexidine digluconate** and therefore is not recommended for use in newborns. The product should not be used by persons allergic to formaldehyde because it contains sodium hydroxymethylglycinate.

Do not disperse of the wipe in the environment after use.

Keep away from dust and store in a cool, dry place, at a temperature not above **25 °C**.

Do not use **KLIMÌ MED** after the expiry date.

The use of **KLIMÌ MED** by children or persons with limited abilities should be supervised by a responsible adult.

Undesirable effects

Occasionally this product may cause a mild and transient burning sensation or mild local irritation.

Should any of the above mentioned reactions occur, discontinue the use of this product.

Interactions with drugs or other medical devices for ophthalmic use

There are no known interactions of the product with the administration of drugs or other medical devices for ophthalmic use.

READ CAREFULLY THE INSTRUCTIONS BEFORE USE OF PRODUCT - FOR EXTERNAL USE ONLY

KEEP OUT OF THE SIGHT AND REACH OF CHILDREN

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SIFI S.p.A.– Via Ercole Patti, 36 – 95025